



*Installing the Touch Screen Alarm/Zone Valve Ethernet Communication Kit 35-3034*

If installing the Ethernet kit in a Tri-Tech alarm/zone valve back box, position the Ethernet kit at the bottom left center of the back box.

1. Turn the power off to the alarm/zone valve box.

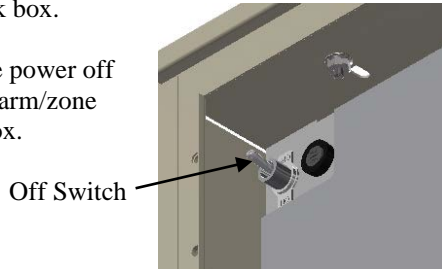


2. Unplug the Ethernet rabbit board from the Base Circuit Board.

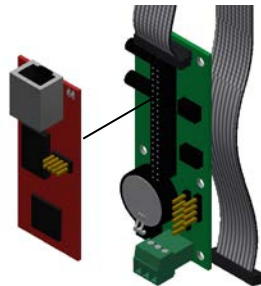


3. Locate the four PEM's stand-offs located in the bottom center of the back box.



4. Using the four short screws supplied, secure the base circuit board to the four PEM's stand-offs, orienting the round battery to the right. Leave the ribbon cable plugged into the base circuit board.

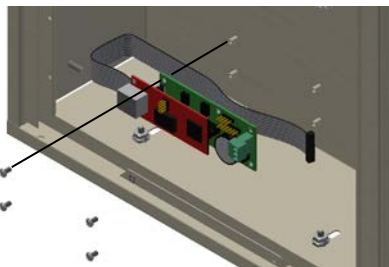


5. Carefully plug the Ethernet Rabbit Board back on the base circuit board, being sure that all pins are engaged. The Ethernet plug socket should be oriented to the left.

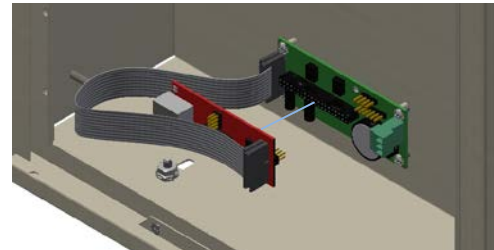

6. Plug the loose end of the ribbon cable into the 35-0009 Button Boards mating socket being sure that the tab on the lug engages the slot in the socket.



7. Plug a Ethernet cable into the socket on the Ethernet Rabbit Board.

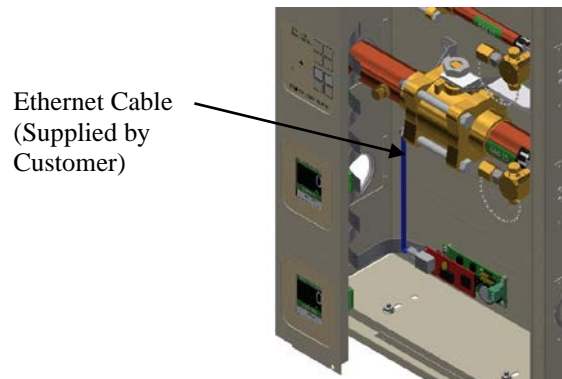

8. Follow the instructions for integrating the Ethernet/Rabbit Board into your network, in the Integra Touch Series Installation & Operation Instruction Manual 99-0502.