

FOR TRI-TECH MODEL NUMBERS:

F4304 Nitrous Oxide F4316 Medical Air F4322 Medical Vacuum F4324A Oxygen

Schrader Outlet (1/4 NPT)


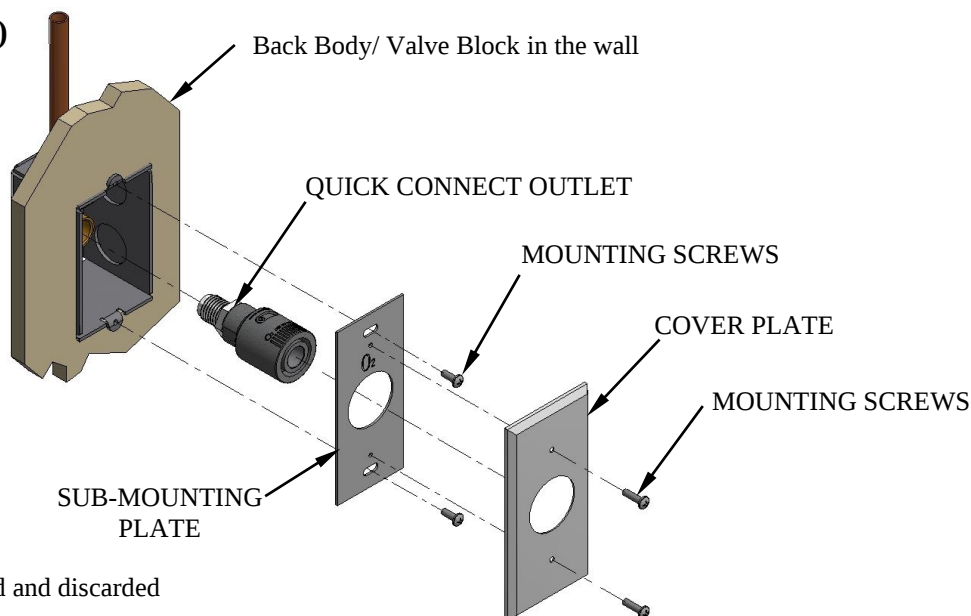


Figure 1
Outlet components to be removed and discarded

Tri-Tech Conversion Outlet

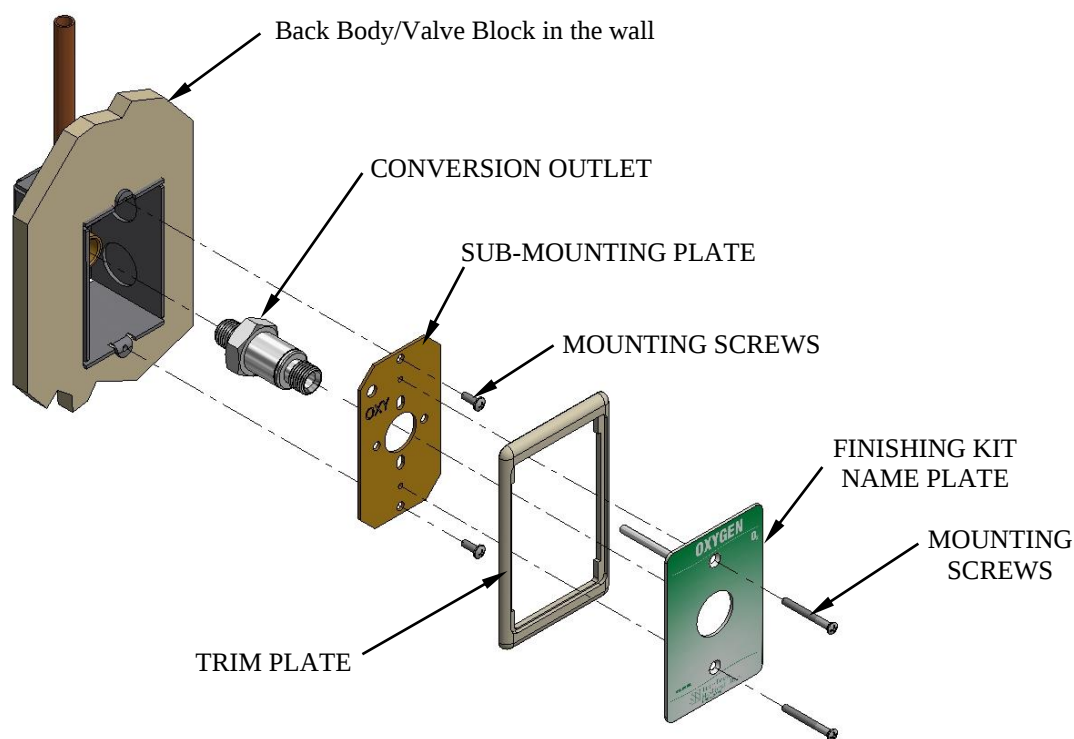

Oxygen Outlet Shown


Figure 2
Installation of the Conversion Outlet Components

Tools Needed: 1- Phillips Screwdriver
1" Hex Wrench

1-Flat Blade Screwdriver
11/16" Hex Wrench


Loctite® (Supplied with Conversion Kit)

Refer to reverse side for detailed instructions

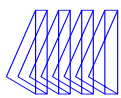

Schrader (1/4 NPT) Series Outlet Disassembly

Close the area zone valve. Warning: Do not close zone valve until all personnel have been advised of the intended service and all systems requiring medical gas are being supplied from portable or alternate supplies. Bleed the residual gas pressure from the outlet.

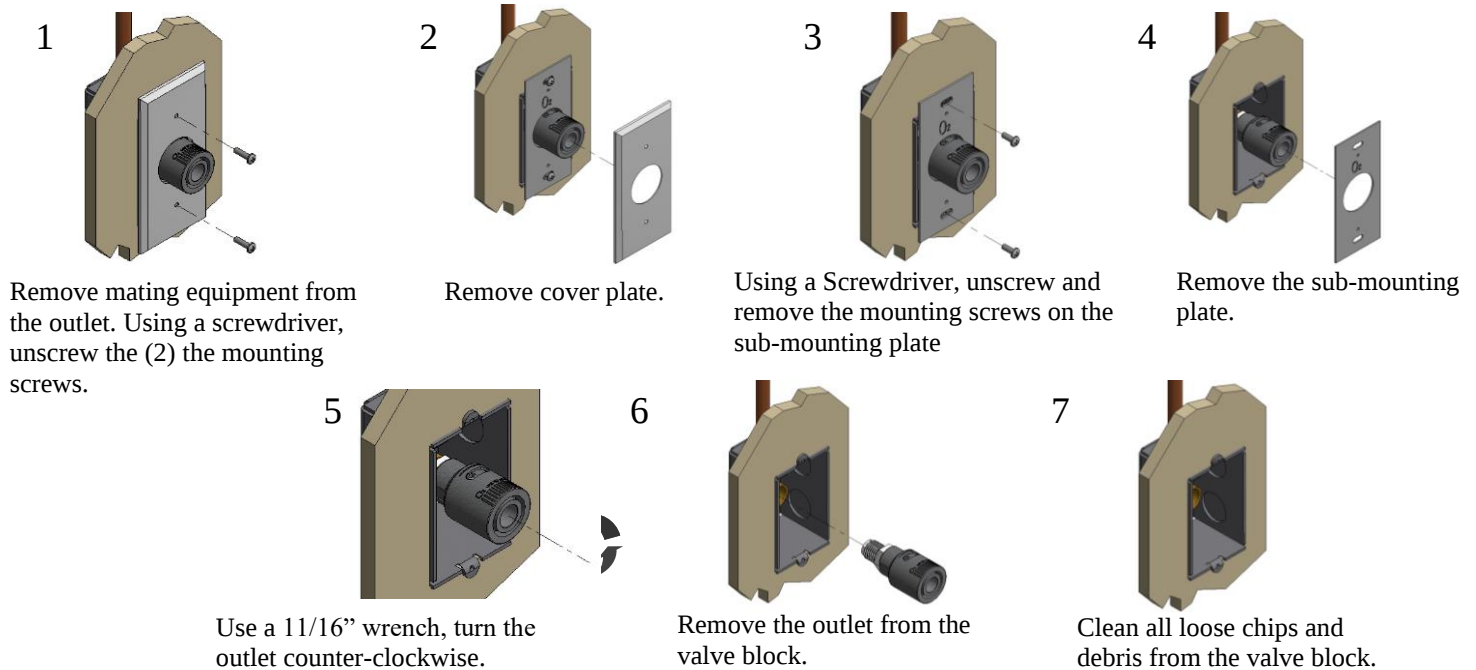

Tri-Tech Conversion Outlet Installation Instructions

