


90 Degree PURITAN STYLE OUTLETS – XP SERIES

Puritan Bennett Style Compatible

Oxygen Outlet Shown

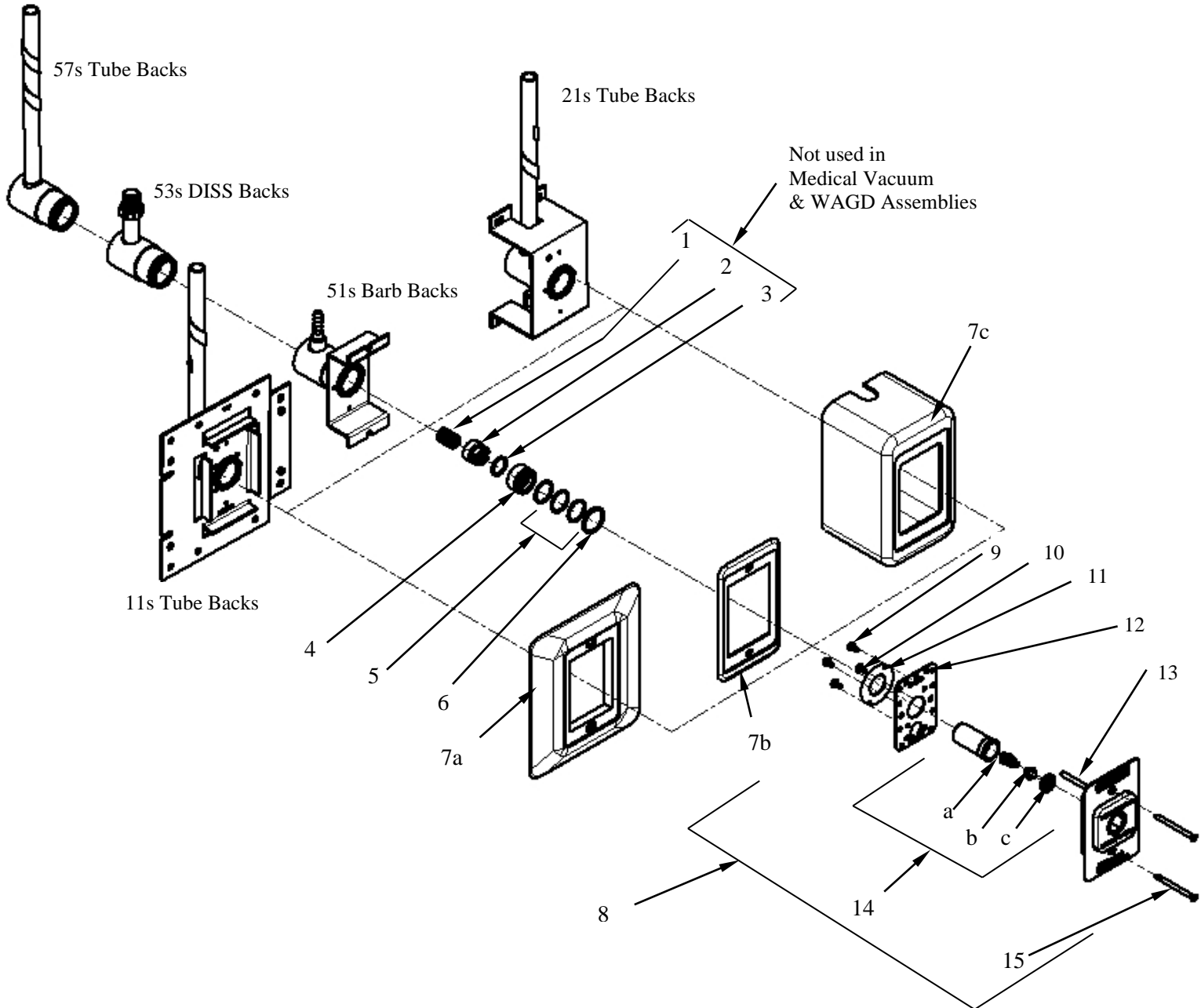

Item 8 Includes Items 9 through 15 in the Finishing Kit

90 Degree Puritan Style Outlets – XP SERIES Bill of Material

Item	Part Number	Description	Quantity	Note
1.	13-1003R	Spring	1	
2.	XC1000-5R	Poppet	1	
3.	11-0050R	Poppet O-Ring	1	A
4.	XC1000-6R	Insert Sleeve	1	
5.	11-0047R	Sleeve O-Ring	3	B
6.	12-1045R	Retaining Ring	1	
7a.	XC1000-9R	Finishing Kit Trim Plate	1	
7b.	XC1000-10R	Trim Plate Cover	1	
7c.	XC2100-3R	Finishing Kit Housing (Surface Mount)	1	
8.	See Below	XP Finishing Kit Assemblies	See Below	
9.	12-3086R	Mounting Screws	2	
10.	12-3087R	Retaining Washer Mounting Screws	2	
11.	XP1000-14R	Retaining Washer	1	
12.	AQ1000-12R	Indexing Housing Cover	1	
13.	XP1000-1R	Indexing Pin	1	
14.	See Below	Front Body Component Kit	See Below	
	(Ref XP1000-13R)	Front Body (only)		
15.	30-0003	Mounting Screw Kit (2) 12-3037	1	

Item 8 - Outlet Finishing Kits (All kits are assembled with Items 9 through 15)

XP1004-A	Nitrous Oxide	1
XP1016-A	Medical Air	1
XP1022-A	Medical Vacuum	1
XP1024-A	Oxygen	1
XP1032-A	WAGD (Waste Anesthetic Gas Disposal)	1

Item 14-XP1000-ERK - Front Body Component Kit (Kit is assembled with Items a through c)

a.	13-1016R	Poppet Spring	1
b.	XP1000-10R	Poppet	1
c.	XP1000-12R	Grommet Washer	1

Secondary Valve Kit

XC1000KR	Secondary Valve Kit for all Gases Except Medical Vacuum & WAGD (Kit Includes Items 1 through 6)	1
XC1022KR	Secondary Valve Kit for Medical Vacuum & WAGD (Kit Includes Items 4 through 6)	1

Note: **A:** Install O-Ring item 3, on Poppet item 2. **B:** Install O-Rings item 5, onto Insert Sleeve item 4.