



**Submittal Data Sheet**

**Features - Benefits:**

- Five year parts and one year labor limited warranty\*
- High intensity LED Lights
- Loud audible alarm buzzer rated at 85 DBA @ 10cm = 4”
- Panel is back mounted
- Durable metal case
- Solid state components
- Plugs into most electrical outlets ♦

\* See Terms and Conditions, Document No. 99-0477, on our website at: <https://tri-techmedical.com> for complete details.



**Model TAV-1** - Shown above  
(With interchangeable plug)

**Specifications:**

Remote Audio/Visual Alarm is specially designed to provide warning of cylinder depletion for a one gas manifold system.

The visual portion of the alarm will change from a “green” indicator for system normal operation to a “red” indicator for system abnormal.

Once the system is re-pressurized, the indicator lights will return to the normal “green” position. When the cylinders are depleted, the alarm buzzer will automatically sound. A reset button conveniently located on the front of the unit will deactivate the alarm buzzer.

The TAV-1 alarm may be used in conjunction with a pressure switch or with a set of dry contact alarm signal points. The alarm is supplied with a transformer (100-240 VAC x 5 VDC) that plugs into an electrical outlet and a 4 1/2 foot cord that goes from the transformer to the alarm. Ideal for industrial, medical and specialty gas use.

**Ordering Information:**

Part Number	Description	Dimensions
TAV-1	Single gas Remote Audio/Visual Alarm	4 5/8” H x 3” W x 1 7/8” D

♦Six different interchangeable plug configurations are provided to adapt the transformer to a variety of International plug patterns.